OtoSensAI

EAR IMAGE CLASSIFIER

Simple Bilingual Guide for Community Health Workers and ASHA Workers

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Training / Evaluation-Period Booklet

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This booklet explains the labels in simple language. It is not a diagnostic or treatment manual. During the current test period, OtoSensAI output must not be used by itself to direct, delay, change, or escalate patient care.

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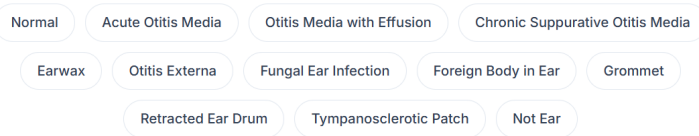
For training and referral support - not for independent diagnosis

Twelve categories, one upload

Each image is labelled as exactly one category and scored against the full list. [What each one means](#)

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Safe capture: clean tip • consent • no names/faces/IDs • never force the otoscope.

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Do not wait for an AI result or reviewer comment

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Simple meaning: The ear canal and eardrum look healthy. No obvious blockage, swelling, hole, discharge, or fluid is seen in the image.

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ROUTINE / सामान्य रेफरल: A “Normal” label does not rule out disease. Refer if pain, discharge, hearing difficulty, fever, dizziness, injury, or symptoms continue.

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Simple meaning: A recent infection behind the eardrum, common in children after a cold. The child may have ear pain, fever, irritability, or reduced hearing.

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SAME DAY / उसी दिन: Arrange same-day medical assessment. Refer urgently if the patient is very ill, pain is severe, fever is high, or there is swelling behind the ear.

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Simple meaning: Fluid is trapped behind an intact eardrum, usually without active pus infection. It mainly causes a blocked ear or reduced hearing.

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EARLY REFERRAL / शीघ्र रेफरल: Refer for hearing assessment if hearing difficulty, speech delay, poor school response, or the problem persists or keeps returning.

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Simple meaning: A long-standing middle-ear infection, usually with a hole in the eardrum and repeated or continuous ear discharge.

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EARLY REFERRAL / शीघ्र रेफरल: Refer to an ENT/secondary centre. Urgent referral is needed for foul-smelling or bloody discharge, severe pain, dizziness, facial weakness, or swelling behind the ear.

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Simple meaning: Wax partly or fully blocks the ear canal or hides the eardrum. It can cause a blocked feeling, itching, ringing, or reduced hearing.

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ROUTINE / सामान्य रेफरल: Refer to a trained clinician for safe removal if the ear is blocked or symptoms are present. Do not remove it with sticks, pins, or ear buds.

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Simple meaning: The skin of the ear canal is inflamed or infected. The ear may hurt when touched or pulled, and there may be itching, swelling, or discharge.

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SAME DAY / उसी दिन: Arrange medical assessment within 24 hours. Refer urgently for severe pain, fever, spreading redness/swelling, or a very unwell patient.

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Simple meaning: Fungus grows in the ear canal. Severe itching, blocked feeling, mild pain, and white, grey, or black debris may occur.

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EARLY REFERRAL / शीघ्र रेफरल: Refer to a clinician/ENT for cleaning and treatment.

Do not put oil, home remedies, or unprescribed drops in the ear.

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Simple meaning: An object or insect is stuck in the ear canal. Common examples are beads, seeds, cotton, insects, and button batteries.

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URGENT / तत्काल: Do not probe or pour liquid into the ear. Button battery, sharp object, bleeding, or severe pain requires immediate emergency/ENT referral; other objects need same-day assessment.

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Simple meaning: A small tube placed by an ENT surgeon through the eardrum to ventilate the middle ear and drain fluid. It may look like a tiny ring.

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ROUTINE / सामान्य रेफरल: Usually this is an expected finding. Refer if there is pain, persistent discharge, bleeding, reduced hearing, or if follow-up is due. Never try to remove it.

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Simple meaning: The eardrum is pulled inward because pressure in the middle ear is not equalising properly. It may cause a blocked ear or reduced hearing.

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EARLY REFERRAL / शीघ्र रेफरल: Refer to ENT/secondary care, especially with hearing loss, repeated discharge, pain, or worsening symptoms.

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Simple meaning: A chalky-white scar patch on the eardrum, usually from an old infection or previous grommet. Small patches may not cause any problem.

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ROUTINE / सामान्य रेफरल: Routine ENT/hearing assessment is needed if there is hearing difficulty, repeated infection, pain, or another abnormal finding.

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Simple meaning: The uploaded image does not clearly show an ear/eardrum. It may be blurred, dark, blocked by wax, mostly canal wall, or a different object/body part.

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SAME DAY / उसी दिन: Do not use this result. Retake one clear image only if safe and within your training. If a good image cannot be obtained, use the usual clinical referral pathway without delay.

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Use symptoms and examination - not the AI label alone

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Record symptoms, duration, ear side, image quality, AI label/confidence, red flags, and referral centre. Follow the approved programme SOP.

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Prepared for training of community-level health workers

1. OtoSensAI official website - platform description, 12 categories, image guidelines, medical and test-period disclaimer.
<https://otosensai.online/>
2. World Health Organization. Primary ear and hearing care: training manual (2023).
<https://www.who.int/publications/i/item/9789240069152>
3. National Health Systems Resource Centre, India. ENT Care Training Manual for ASHA.
<https://nhsrcindia.org/sites/default/files/2021-12/ENT%20Care%20Training%20Manual%20for%20ASHA.pdf>
4. Ministry of Health and Family Welfare, Government of India. Operational Guidelines for ENT Care at Health and Wellness Centres.
https://aam.mohfw.gov.in/download/document/Operational_Guidelines_Ear%2C_Nose_and_Throat_%28ENT%29_Care_at_HWCs.pdf
5. Merck Manual Professional Edition. Ear foreign body removal - button batteries require prompt removal.
<https://www.merckmanuals.com/professional/ear-nose-and-throat-disorders/how-to-do-ear-procedures/how-to-remove-a-foreign-body-from-the-external-ear>

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The OtoSensAI website states that the platform is being tested under CDSCO Form MD-13 and has not received market approval. During the test period, outputs are investigational and must not direct patient care. Use only within the approved study/programme and under supervision.

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Clinical review: ENT specialist / programme medical officer

For feedback: voxmedai@gmail.com